Tubes Radiogènes à Anode Tournante
Drehanoden - Röntgenröhre
Tubos de Rayos-X con Ánodo Giratorio

MCS-6074/B-580H

Note: Document originally drafted in the English language.

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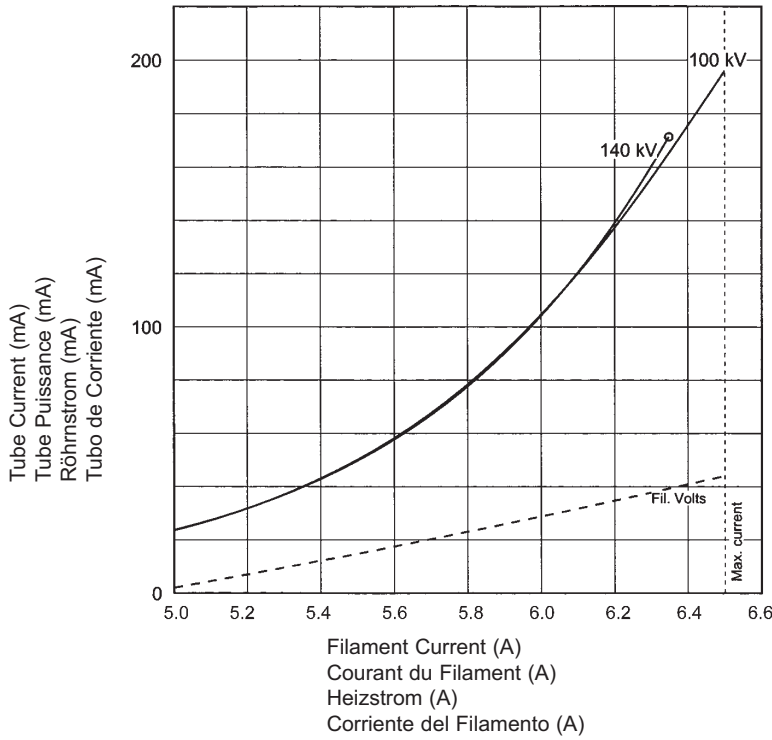
Manufactured by Varian Medical Systems
Fabrique par Varian Medical Systems
Hergestellt von Varian Medical Systems
Fabricado por Varian Medical Systems


Specifications subject to change without notice.
Spécifications susceptibles d'être modifiées sans préavis.
Technische Daten ohne Gewähr.
Especificaciones sujetas a cambio sin previo aviso.

Caractéristiques d'Émission du Filament CEI 60613
Kathoden - Emissionskennlinien IEC 60613
Características de Emisión del Catodo IEC 60613

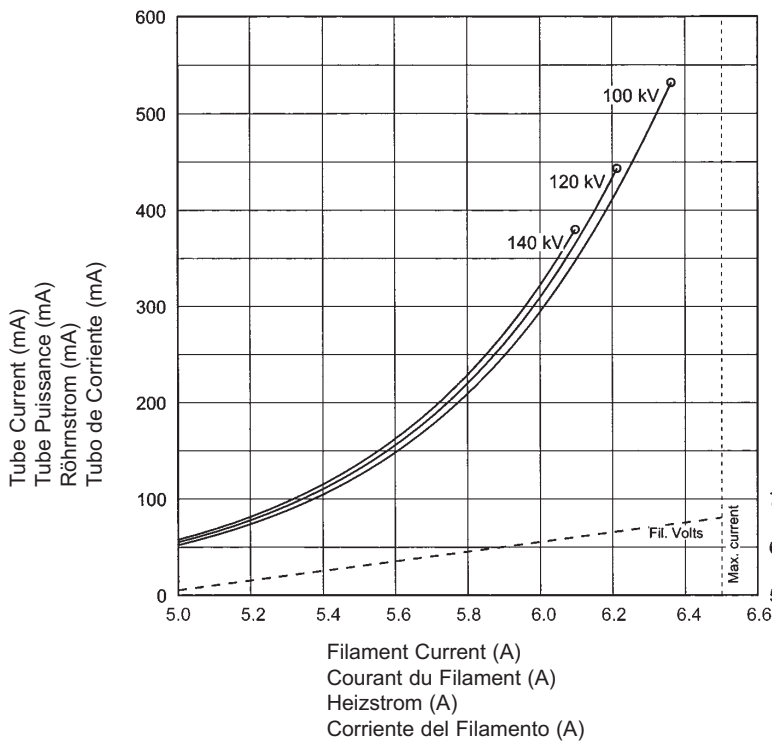THREE PHASE EMISSION ($\pm .15$ A)
MCS-6074 0.9mm x 0.7mm

THREE PHASE EMISSION ($\pm .15$ A)
MCS-6074 1.2mm x 1.2mm

Le Gaine B-580H

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Encaje de B-580H


Maximum Peak Voltage	140 kV
Anode to Ground	70 kV
Cathode to Ground	70 kV
Maximum X-ray Tube Assembly Heat Content	4.0 MJ (5.5 MHU)
Maximum Continuous Heat Dissipation (max. housing temperature 78°C) (Includes stator heat)	3.7 kW (5.0 kHU/sec)
Focal Point Position (Central Ray) Within 1mm (X, Y Direction from the center of radiation port.)	
X-Ray Tube Assembly	
Permanent filtration	0.32 mm Al @ 75 kV IEC 60522
with Gantry Plate	4.5 mm Al @ 75 kV IEC 60522
Loading Factors for Leakage Radiation	140 kV, 26.4 mA
Ambient Air Temperature Limits for Operation	5°C to 35°C
Temperature Limits for Storage and Transport	-34°C to +60°C
Humidity	20% - 70%
Weight - Tube and Housing	84 kg (185 lbs)
IEC Classification	Class I
Safety Devices - Thermal Switch	
Normally Closed Contact	Opening at 77°C ±3°C
Pressure Switch	Normally Closed Contact Contact opens with adequate oil flow.

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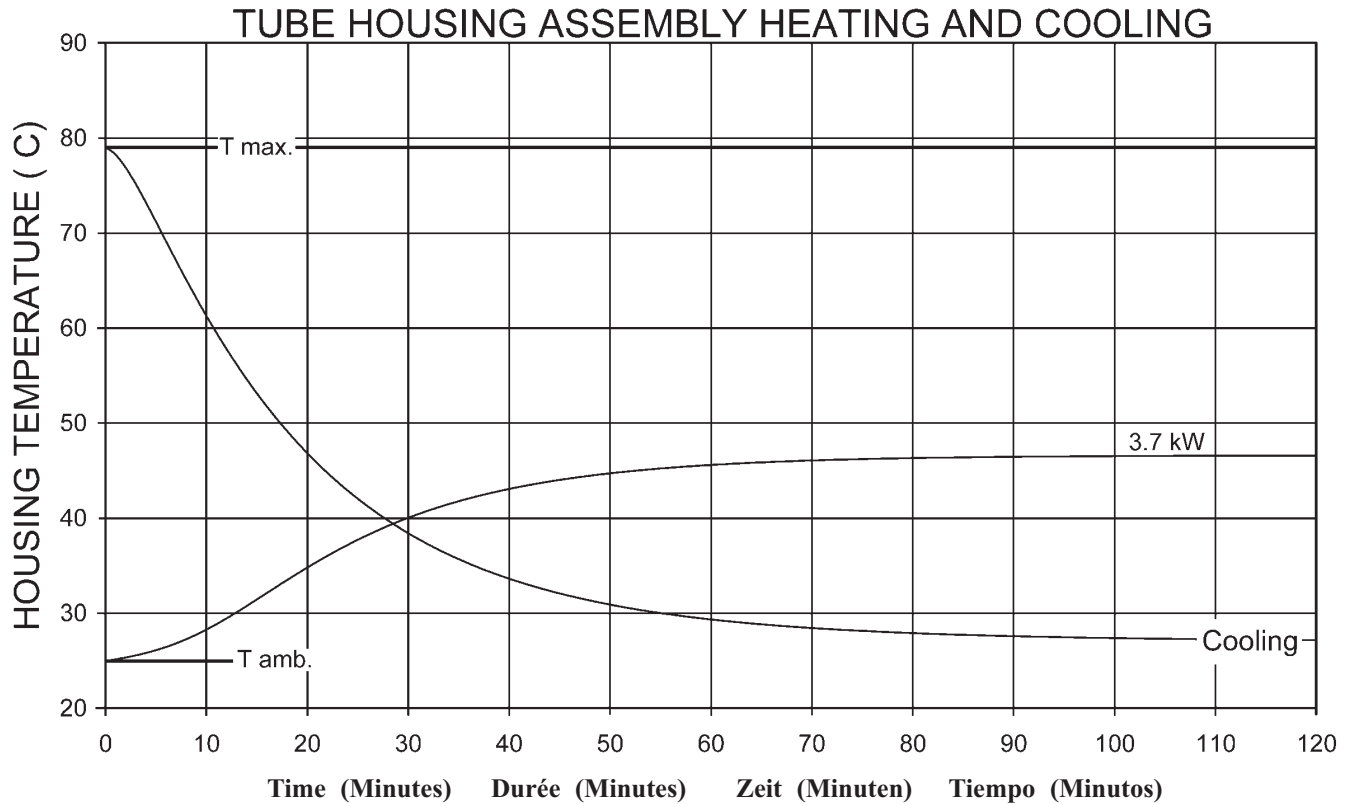Note:

- Heat inputs into housing include tube power, filament power, and stator power.
- Heating curves based on no restrictions of natural convection around tube housing assembly.
- Heating and cooling curves reflect maximum tube performance. Tube operation is ultimately limited by system software control.

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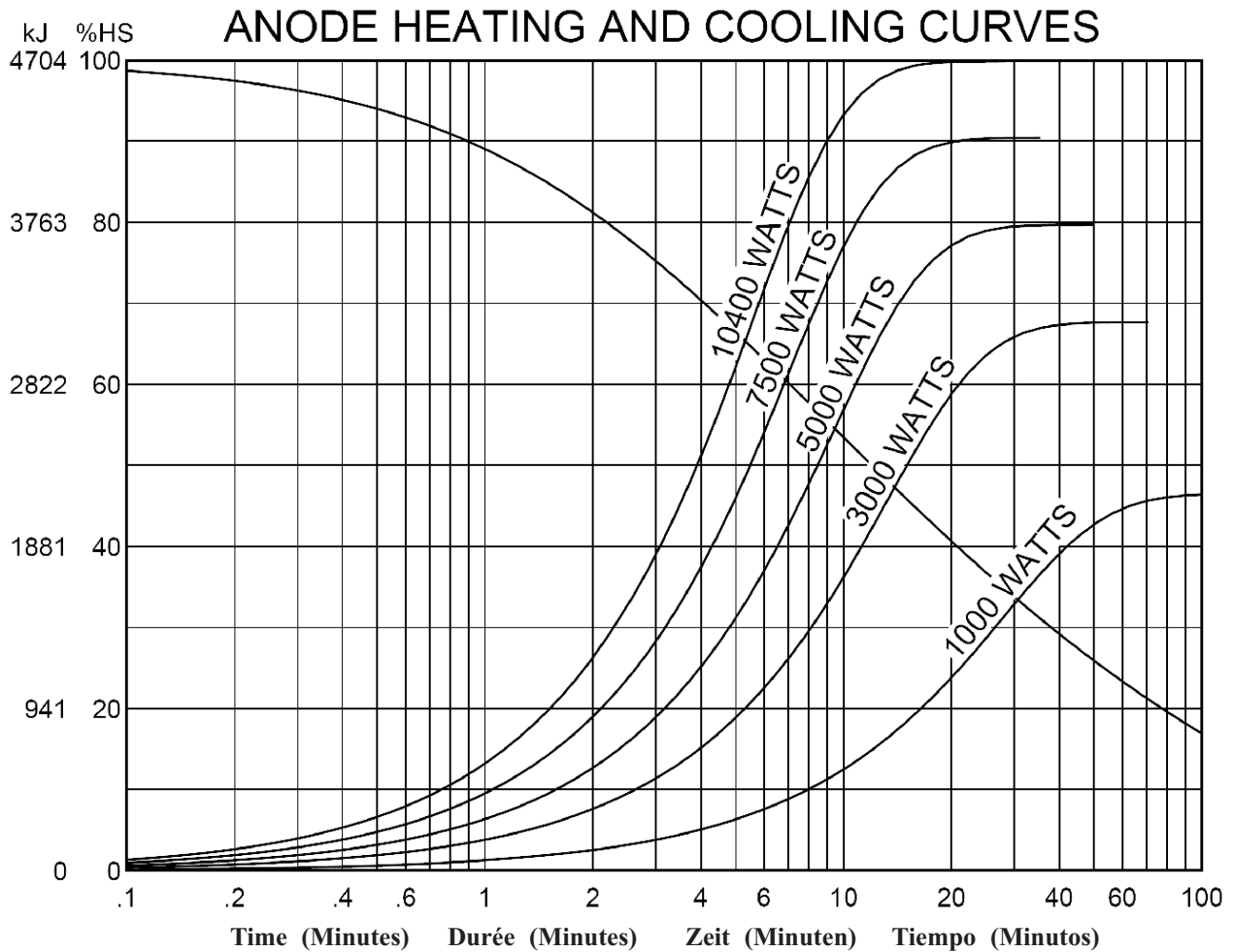
Note:

1. Heating and cooling curves reflect maximum tube performance. Tube operation is ultimately limited by system software control.

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